


Osstell Beacon

How to register your instrument


1 Create your OsstellConnect account

Go to www.osstellconnect.com/register
Follow the steps.

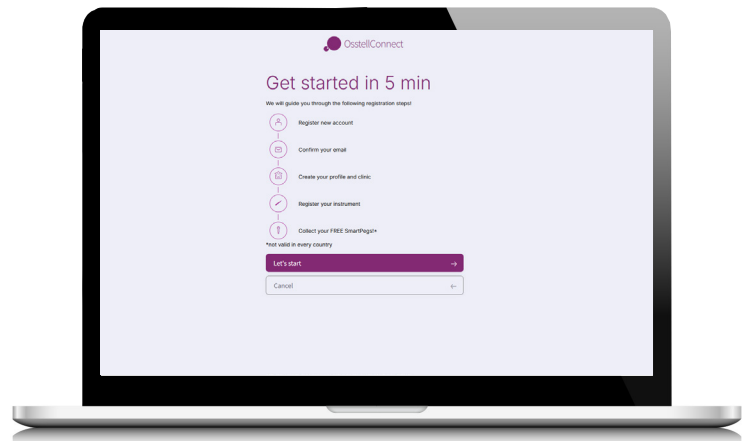

2 Select Osstell Beacon


Choose **Osstell Beacon** as your instrument.

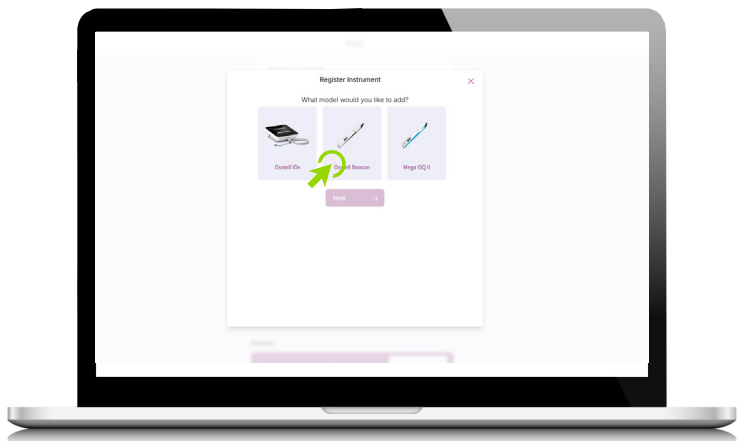

3

Download the gateway

Download the gateway for Windows or Mac. Follow the installation instructions on screen.

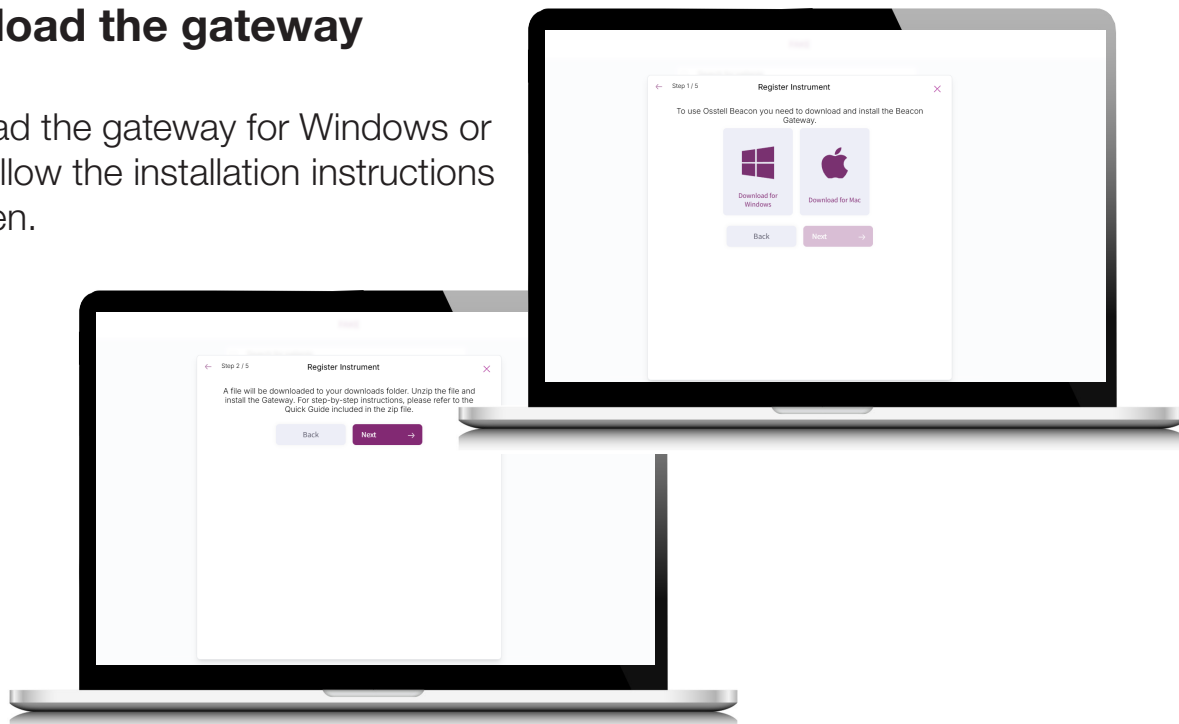

4

Register your instrument



Enter:

- **Serial number** – found on the back of the instrument (A)
- **Registration code** – shown on the lower display when you lift the instrument (B)

Please note:

It may take a few seconds for the instrument to connect.

When connected, this symbol will appear in the upper display: ((•••))

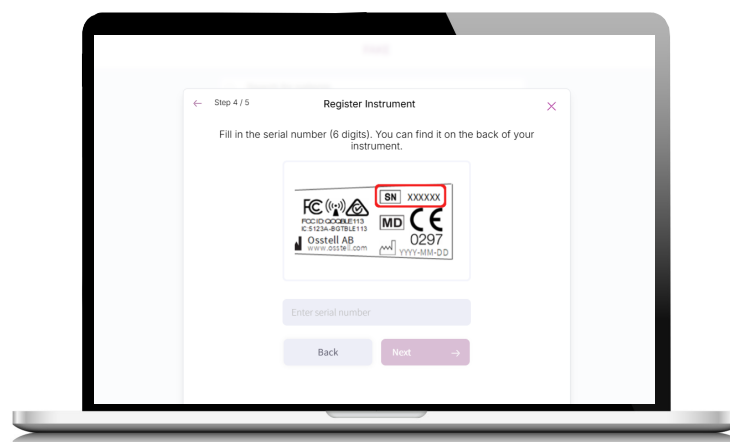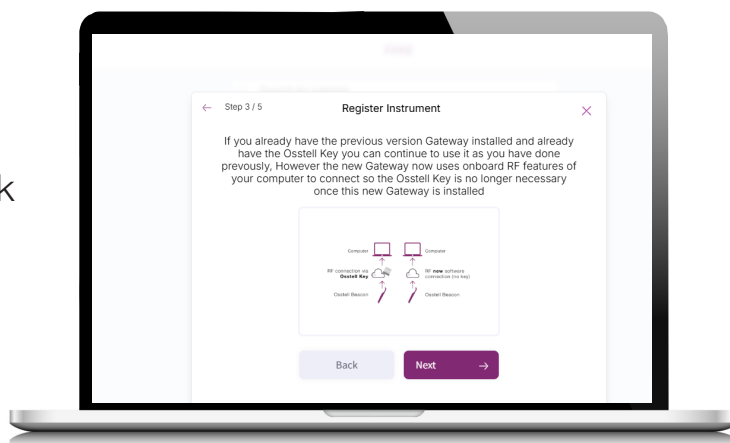


You are connected!

Your Osstell Beacon is now registered to OsstellConnect.
You can now start storing and reviewing your ISQ measurements.software.

Our token of appreciation

As part of the OsstellConnect community, you can collect your free single-use SmartPegs.

- Click “Collect SmartPegs”
- Select your preferred SmartPeg type
- Enter your delivery details

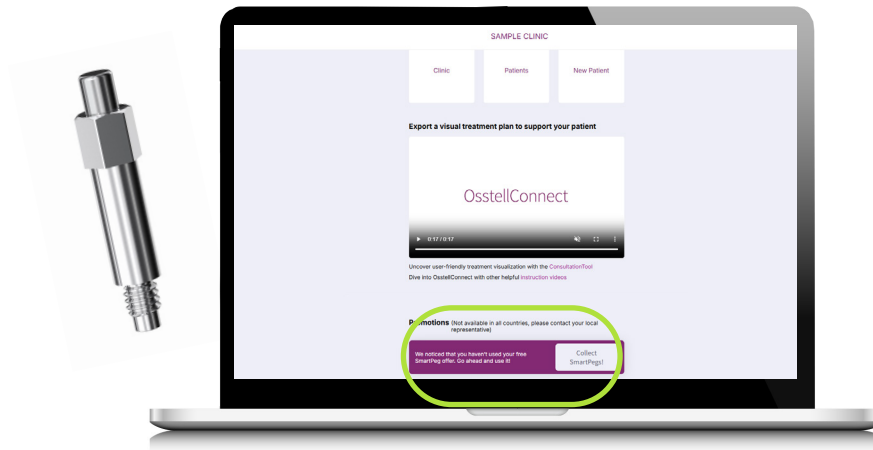

Please note: delivery time may be slightly longer than when ordering from your local reseller.